

Bipolar multifunctional forceps

sterile · pack of 12 · fast · bloodless · clean




The bipolar multifunctional forceps are part of the ideal accessories and are especially developed for the removal of **pedunculated warts or fibroma pendulans**. It can be connected to all electrosurgical units with bipolar function.

During removal of the pedunculated warts or fibroma pendulans no cutting is required and a surgical suture is no longer necessary. The treatment is bloodless and does not result in any scars.



No local anaesthesia is required.

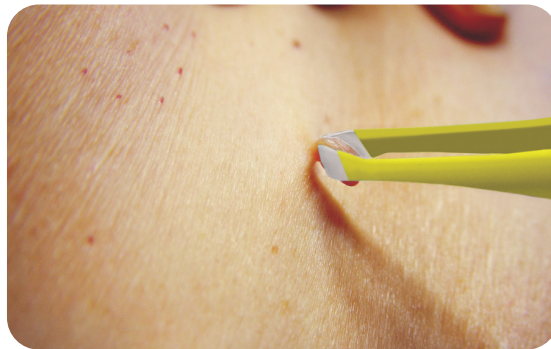

The pedunculated wart or fibroma pendulans is held at the base with the forceps, and is separated from the skin by means of coagulation effected by a pulse trigger over the footswitch.



Right after treatment






After healing



No after-bleeding occurs during this simple and fast method, as all feeding blood vessels are being coagulated. Due to immediate coagulation while cutting, the infection caused by viruses of pedunculated warts or fibroma pendulans is prevented.

Bipolar multifunctional forceps

sterile · pack of 12 · fast · bloodless · clean

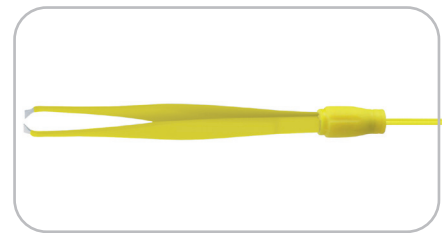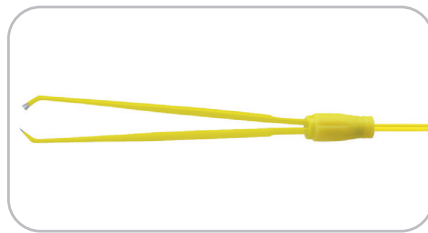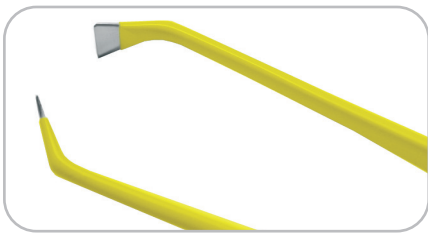

- At both ends of the forceps grip there are cutting edges, which are inwardly angled facing each other. Thereby the coagulation, the cutting process and the removal are carried out simultaneously.
- It is of additional advantage that due to the inwardly angled cutting edges the operating speed is higher.
- Furthermore, the risk of injury to the patient or surgeon is significantly lower compared to using a scalpel or an electrode.
- Single use, sterile.



Bipolar multifunctional forceps, inside angled, length 130 mm, wide 5 mm, sterile, 3 m cable, pack of 12, single use

Art.-No. 140-115-013-YC

